



Conform Cushion™

To secure your ConForm Cushion to a wheelchair:

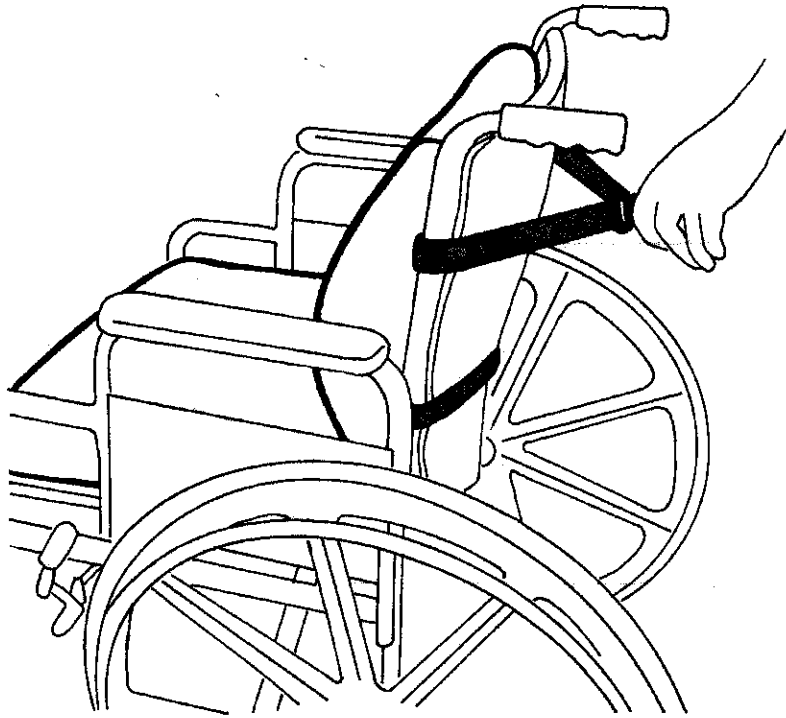

Wrap the straps around the back of the wheelchair.

Slide the ends of the straps through the "D" rings and pull tight.

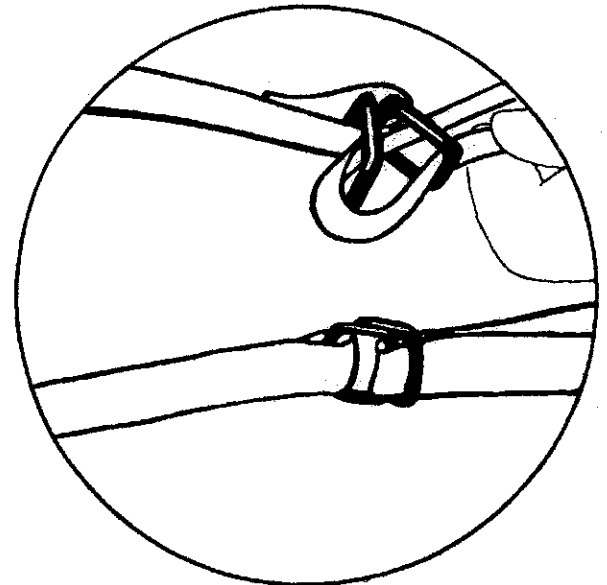

To Clean:

Remove the cover from the foam cushion. The cover may be machine washed in warm water on the gentle cycle and tumble dried on a low setting.

The foam cushion may be wiped clean with a damp cloth. Please do not submerge the cushion in water or puncture the skin on the foam.



Sammons Preston Rolyan
An AbilityOne Company
4 Sammons Court
Bolingbrook, IL 60440-4989
Phone: 1-800-323-5547
In Canada Phone: 1-800-665-9200